

MRI

Magnetic Resonance Imaging

CRHF Technical Services Standard Letter

Rev. 1.0, September 2016, © Medtronic 2016

The information provided in this letter is for healthcare providers and Medtronic representatives and applies to the following Medtronic device models:

Model number, model name, description (Non-MRI Conditional devices)

ADDR01/03/06 Adapta, dual chamber IPG

ADDRS1 Adapta S, dual chamber IPG

ADDRL1 Adapta Large, dual chamber IPG

ADD01 Adapta, dual chamber IPG

ADVDD01 Adapta, VDD IPG

ADSR01/03/06 Adapta, single chamber IPG

VEDR01 Versa, dual chamber IPG

SEDR01 Sensia, dual chamber IPG

SEDRL1 Sensia Large, dual chamber IPG

SED01 Sensia, dual chamber IPG

SESR01 Sensia, single chamber IPG

SES01 Sensia, single chamber IPG

REDR01/RED01 Relia, dual chamber IPG

RESR01/RES01 Relia, single chamber IPG

REVDD01 Relia, VDD IPG

Magnetic resonance imaging (MRI) – Do not use magnetic resonance imaging (MRI) on patients who have an implanted device. MRI may cause damage to the device and/or serious injury to the patient.

Labeling

Adapta/Versa/Sensia/Relia Pacemaker Reference Guide. Manual Document Number: M939295A001 REV. D, www.medtronic.com/manuals.

PATIENT MANAGEMENT GUIDANCE

This document is useful to health care professionals who perform medical procedures on patients with Medtronic implanted cardiac device systems and who consult with the patients' cardiologists.

How to contact U.S. CRHF Technical Services:
Pacemakers: (800) 505-4636, ICDs: (800) 723-4636, Instruments: (800) 638-1991.
Email: tshelp@medtronic.com

This information is verified for devices approved in the U.S. and may differ by country. For product-specific information on device operation and indications for use, reference the appropriate product labeling.

MRI

Continued

Additional comments

For further information please contact the following:

- **Technical questions:** Medtronic Technical Services can answer additional questions regarding these device operations.
Device labeling: For additional device-specific guidance, consult the labeling associated with the device available on the Medtronic Manual Library website at www.medtronic.com/manuals.
- **Patient questions:** Patients who have questions can contact Medtronic Heart Rhythm Patient Services at 1-800-551-5544, Option 3, email at pshelp@medtronic.com, or see www.medtronic.com/rhythms for a variety of resources.

How to contact U.S. CRHF Technical Services:
Pacemakers: (800) 505-4636, ICDs: (800) 723-4636, Instruments: (800) 638-1991.
Email: tshelp@medtronic.com

This information is verified for devices approved in the U.S. and may differ by country. For product-specific information on device operation and indications for use, reference the appropriate product labeling.

Page 2 of 2

Medtronic